




How to change Omnicell labels.

Updated: March 4, 2022

Objective: Replace empty label Omnicell label roll

- Depending on the Omnicell type, printers accessed differently, see image.
 - Full-size Omnicell: printers are located behind the keyboard. Pull to Open.
 - Half-size or Table-top Omnicell: printers are to the right of the keyboard. Lift to Open.

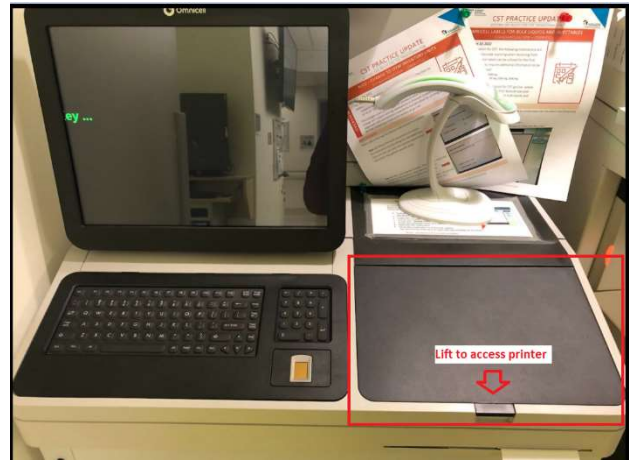

- Follow instructions inside the printer panel.

Full-size Omnicell:



Half-size or Table-top Omnicell:



3. In case of a label jam:

- Open the printer panel.
- Click little green lever to release the tension on the paper roll.
- VERY carefully pull out all jammed paper. If you rip it you may need tweezers.
- Tear off all damaged paper from the roll.
- Re-thread paper using instructions for changing paper roll.
- Remember to click the green lever back!
- If you are unable to fix the jam for some reason and the cabinet is still not running properly you will have no choice but to call pharmacy.

4. Omnicell Labels and Receipt paper can be ordered from the Stores Catalogue.

❖ *Unit Clerks can enter "Omnicell" in the Search bar*

Labels:

LABEL 3.1 X 1.0 A 2.25 WD LABEL ROLL 3.1 ODX1.0IDX2.25WD LABEL 3.1 FOR OMNICELL	714683	OMNICELL INC	95-6084	95-6084	OMNICELL
--	--------	--------------	---------	---------	----------

Receipt Paper:

PAPER THERMAL 85 MM XT PAPER 92M ROLL 85 MM WIDE PAPER THERMAL 85 MM XT	714683	OMNICELL INC	95-6096	95-6096	OMNICELL
---	--------	--------------	---------	---------	----------

❖ *If your unit needs a Label roll urgently, please contact Pharmacy and they can tube up a roll.*