

HemoCue Hb 201+: Appendix H - Loaner Instrument and Replacement

Operator Refers to:

- HemoCue Hb 201+: Quick Reference Guide
- HemoCue Hb 201+: Appendix E - Troubleshooting Guide
-

Note: For any Error Code or instrument failure if the problem persists, Operator is to:

1. Perform troubleshooting steps as listed beside the Error Code/Problem.
2. Repeat the Hemoglobin test with a new microcuvette when possible.
3. If unable to obtain a Hemoglobin result or perform a QC check obtain a replacement/loaner HemoCue Hb 201+ Analyzer from the lab.
4. Contact Point of Care Testing (POCT) Technologist with questions or concerns.

POCT Technologist Contact:

Monday to Friday: Phone 604-875-2345. Local 7521 Room 2J8
Pager 41-02190.

After hours contact Routine Hematology:
Phone 604-875-2345. Local 7821 Room 2J17

HEMATOLOGY TECHNOLOGIST

Provide assistance to the POCT HemoCue Hb 201+ Operator by:

5. Replacing the faulty/defective HemoCue Hb 201+ with one of the "POCT LOANER" Analyzers. Room 2J8.
- Place a note onto the defective HemoCue Hb 201+ for the POCT technologist.
6. If Microcuvettes are required by the Clinic/Program, a supply is provided in the bin with the HemoCue Hb 201+ analyzers.
 7. Quality Control (QC) is provided by POCT. QC vials – Levels 1, 2 & 3 are available in the Vitros Specimen Fridge Room 2J17.
 8. Document all loaner or supply transactions with the ward. Include date, time, ward, meter #, supplies given, technologist name on the Utilization Record Form B.

References:

HemoCue Hb 201+ Operating Manual. HemoCue AB, Box 1204 SE-262 23 Angelholm, Sweden. info@hemocue.se www.hemocue.com

HemoCue Hb 201 Microcuvettes, Package Insert. HemoCue AB, Box 1204 SE-262 23 Angelholm, Sweden. info@hemocue.se www.hemocue.com

Eurotrol HemoTrol, Assayed Controls for In Vitro Diagnostic Use, Package Insert. Eurotrol B.V. Postbus 722 6710 BS Ede, The Netherlands office@eurotrol.com www.eurotrol.com

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	Jun 2012	Elvira Kozak	Dr. Cathy Halstead
1.1	Minor	Document title and number change. Upload to QMS document control	28 Dec 2016		Dr. Benjamin Jung

Attention: This document is published on the BCCW SharePoint website

Revisions made to this document require an update to the corresponding document published on BCCW SharePoint website