

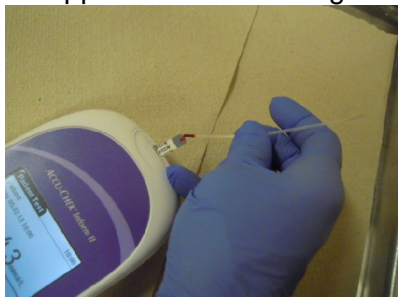



## Accu-Chek Inform II: Microcollection Pipette - Quick Reference Guide

<b>Use of the Micro-collection pipette</b>	
<b>1</b>	<p>Micro – collection pipette can be used for collecting a drop of whole blood for a heel (or finger) puncture. (The pipettes from Poly-Pipets Inc. Vendor Item ID 40010NL058 is ordered and supplied by NICU).</p> 
<b>2</b>	<p>Squeeze micro-collection pipette to draw whole blood from heel puncture into pipette</p> 
<b>3</b>	<p>Release hold on micro-collection pipette. Squeeze to dispense 0.6 uL whole blood for application to front edge of test strip</p> 
<b>4</b>	<p>Alternate is to apply whole blood directly from the heel (or finger) micro-puncture site</p> 



## REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	01 Mar 2013	Elvira Kozak	Dr. Cathy Halstead
1.1		Revision	21 Nov 2014	Elvira Kozak	
1.2	Minor	Document title and number change. Upload to QMS document control. Format Change	27 Dec 2016		Dr. Benjamin Jung
1.3	Minor	Updated micro collection pipette supply	Calvin Lee	July 2, 2019	

**Attention: This document is published on the ePOPS website**

**Revisions made to this document require an update to the corresponding document published on ePOPS website**