

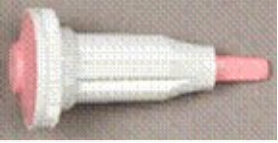


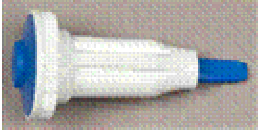


Accu-Chek Inform II: Ordering Supplies and Lancet Use - Quick Reference Guide



Purpose: This procedure provides instructions for use of lancets, meter cleaning and disinfection, and supply ordering information.

Supply list and ordering information:			
Item	Item Photo	Stores Item #	Item Use /Comment
Quality Control Solution Level 1 & Level 2		<i>n/a</i> POCT Office 2J8	Open vial expiry is 3 months
ACCU-CHEK Inform II Test Strips		# 00008422	
Safe-T-Lance – PINK	 1.2 mm depth x 1.5 mm width	# 00083521	Neonate / Infant Heel All weights

PHSA Laboratories CW Site - Point of Care
Title: CWPC_GLU_0105 Accu-Chek Inform II Ordering Supplies Lancet Use - Quick Reference Guide

<p>Safe-T-Lance – Blue</p>	 1.4 mm depth X 25 g	<p># 00083519</p>	<p>Pediatric Finger > 6 months & > 5 kg</p>
<p>OneTouch® SureSoft</p>	 1.8 mm depth X 28 g	<p>00028027</p>	<p>Finger > 8 yr.</p>
<p>CaviWipes (Metrex) 10" x 12" size XL 6" x 6 3/4" size Reg</p>		<p>XL size 00085046 regular size 00010444</p>	<p>Caviwipes is the only cleaning and disinfection product approved by PHSA for use with the AccuChek Inform II Glucose Meter, Base Unit & Tote</p>

Procedure:

	Action	Related Documents
	<p>Meter Cleaning and Disinfection</p> <ul style="list-style-type: none"> • To be performed after each patient use • Wipe once to clean • Wipe twice to disinfect • Use gloves 	
	Lancet Directions	
1	Wear gloves	
2	Choose appropriate puncture site and lancet device for age and weight	
3	Clean and disinfect site. Air dry.	
4	Hold the patient heel or finger firmly to prevent sudden movement.	

Medical Approval: Dr Benjamin Jung Version: 1.2 Folder Name: CW\Point of Care\Whole Blood Glucose This is a controlled document for CW use only. Any printed copies are uncontrolled unless specified. Please refer to Lab QMS	Medical Approval Date: 27 Dec 2016 Implementation Date: 1/4/2017 2:25:33 PM
---	--

PHSA Laboratories CW Site - Point of Care
Title: CWPC_GLU_0105 Accu-Chek Inform II Ordering Supplies Lancet Use - Quick Reference Guide

	Action	Related Documents
5	Turn and twist off the protective cover. Dispose.	
6	Position lancet firmly on disinfected site. Pull skin taut in puncture area.	
7	Press colored button or press lever firmly to activate.	
8	Release button or lever. Lancet automatically retracts	
9	Discard in sharps container.	
10	Wipe away first drop of blood.	
11	Apply sample to test strip.	

For additional information Contact:

Point of Care Testing (POCT) Office: Monday -Friday: 604-875-2345 ext: 7521
Laboratory @ C&W: After hours/holidays: 604-875-2345 ext: 7820
AccuChek Inform II Glucose Meter PHSA Point of Care Testing Rm.2J8

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	01 Mar 2013	Elvira Kozak	Dr. Cathy Halstead
1.1		Revision	22 Oct 2015	Elvira Kozak	
1.2	Minor	Document title and number change. Upload to QMS document control Format change	27 Dec 2016		Dr. Benjamin Jung

Attention: This document is published on the BCCW SharePoint website

Revisions made to this document require an update to the corresponding document published on BCCW SharePoint website

Medical Approval: Dr Benjamin Jung Version: 1.2 Folder Name: CW\Point of Care\Whole Blood Glucose This is a controlled document for CW use only. Any printed copies are uncontrolled unless specified. Please refer to Lab QMS	Medical Approval Date: 27 Dec 2016 Implementation Date: 1/4/2017 2:25:33 PM
---	--