


CoaguChek Outpatient Clinic: Appendix D - CoaguChek XS Plus Battery Replacement

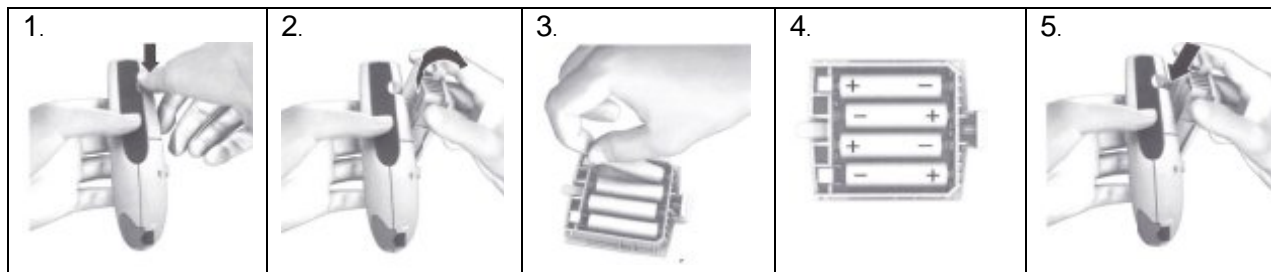

1. Monitor is turned off. Hold monitor so that the tab on the battery compartment cover points upward.
2. Press the tab on the battery compartment cover towards the center of the monitor. Lift the cover to the side.
3. Slide the battery compartment cover upward and remove.
4. Insert four [4] batteries as indicated. Use only alkali-manganese 1.5 V "AA" batteries.
5. Close the battery compartment.

REFERENCES

- CoaguChek® XS Plus Operator Manual, Roche Diagnostics GmbH D-68298 Mannheim, Germany 2009-11
- CoaguChek® XS PRO Operator Manual, Roche Diagnostics, 201 Boul, Armand-Frappier, Laval, Quebec, H7V 4A2 Canada 2010-10
- CoaguChek® XS Operator Manual, Roche Diagnostics, 201 Boul, Armand-Frappier, Laval, Quebec, H7V 4A2 Canada 2011-03

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	Dec 2013	Elvira Kozak	Dr. Cathy Halstead
1.1	Minor	Document title and number change. Upload to QMS document control	28 Dec 2016		Dr. Benjamin Jung

Attention: This document is published on the BCCW SharePoint website

Revisions made to this document require an update to the corresponding document published on BCCW SharePoint website

Medical Approval: Dr Benjamin Jung
Version: 1.1
Folder Name: CW\Point of Care\PT INR

Medical Approval Date: 28 Dec 2016
Implementation Date: 1/3/2017 5:40:56 PM

This is a controlled document for CW use only. Any printed copies are uncontrolled unless specified. Please refer to Lab QMS