

CoaguChek Outpatient Clinic: Appendix F - Meter Set Up

Set the Units



Meter is in Main Menu:

1. Touch the SETUP button.
2. Touch the SCREEN button.
3. Touch the UNITS button.
4. Touch the button to select INR.
5. Touch to save the selection.
6. Touch to return to Previous Screen.

Set the Language



Meter is in Main Menu:

1. Touch the SETUP button.
2. Touch the SCREEN button.
3. Touch the LANGUAGE button.
4. Touch the button to select ENGLISH (US).
5. Touch to save the selection.
6. Touch to return to Previous Screen.

Set the Beep Tone




Meter is in Main Menu:

1. Touch the SETUP button.
2. Touch the OPTIONS button.
3. Touch the BEEP TONE button.
4. Touch the button to select MODERATE, for the beep tone.
5. Touch the button to select ON, for the key beep.
6. Touch to save the selection.
7. Touch to return to Previous Screen.

Note: In the diagram the meter is set to give no beep tone.

Set the Auto Off

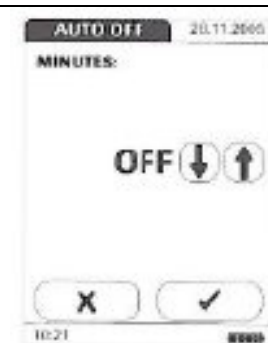

Meter is in Main Menu:

1. Touch the SETUP button.
2. Touch the OPTIONS button.
3. Touch the AUTO OFF button.
4. Touch the arrow buttons to select from OFF to Sixty minutes. *Set the meter for an Auto Off of five [5] minutes.* (Meter will turn itself off automatically if it has not been used for five [5] minutes).
5. Touch to save the selection.
6. Touch to return to Previous Screen.

REFERENCES

- CoaguChek® XS Plus Operator Manual, Roche Diagnostics GmbH
D-68298 Mannheim, Germany 2009-11
- CoaguChek® XS PRO Operator Manual, Roche Diagnostics,
201 Boul, Armand-Frappier, Laval, Quebec, H7V 4A2 Canada 2010-10
- CoaguChek® XS Operator Manual, Roche Diagnostics,
201 Boul, Armand-Frappier, Laval, Quebec, H7V 4A2 Canada 2011-03
- CoaguChek® XS PT Test Strip Directional Insert, Roche Diagnostics,
201 Boul, Armand-Frappier, Laval, Quebec, H7V 4A2 Canada 2010-05
- CoaguChek® XS PT Controls Directional Insert, Roche Diagnostics,
201 Boul, Armand-Frappier, Laval, Quebec, H7V 4A2 Canada 2009-12

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	Dec 2013	Elvira Kozak	Dr. Cathy Halstead
1.1	Minor	Document title and number change. Upload to QMS document control	28 Dec 2016		Dr. Benjamin Jung

Attention: This document is published on the BCCW SharePoint website

Revisions made to this document require an update to the corresponding document published on BCCW SharePoint website