

DCA Vantage HbA1C: Appendix D - Instrument Settings

Access Settings from MENU on the Home Page.					
Control Tests		Gives the QC Status			
System Maintenance		Gives the Maintenance Schedule and Status.			
System Settings					
Sound Volume	Medium				
Display	Contrast 5				
Additional Settings Requires Supervisor ID – Restricted Access.	Patient Tests	Data Entry	Sample ID – <i>Lab Use - Sunquest CID #</i> Patient ID – <i>Clinic Use – Pt. MRN</i> Last Name, First Name User ID Comment 1		
		Reference Range	HA1c 4 – 6 %		
		Patient Recall List	Last Name		
	Control Tests	Data Entry	User ID		
		Reminders	HA1c Normal	Daily	
			HA1c Abnormal	Daily	
		Other Manufacturer QC	Use of other manufacturer QC		
	Maintenance	Change Air Filter	Quarterly		
		Clean Barcode Window	Monthly		
		Clean Cartridge Compartment	Quarterly		
		Run Optical PD Tests	Quarterly		
	General	Language	English		
		Time	24 HR		
		Date	DD.MM.YYY		
		Sequence #	Resets Daily		
		System Access	Restricted	POCT	1001
			Supervisor	MH METAB	1003
				CH DDC	1005
		Test Lockout	No QC Lockout for Patient Testing		
		Settings Wizard	With each Power ON		
Storage Mode	Automatic Deletion of Tests				
Printers	Onboard Printer	Enabled			
	External Printer	OFF			
	Report Header	MH METABOLIC CLINIC			
External Barcode Reader	Disabled				
HA1c Reporting Units	1° NGSP				
System Tests	User System Tests	Not Enabled			
	Service System Tests	Service Access Only with ID			
	Optical Tests	Precision and Drift.			
	Calibrate Touchscreen	Follow the touch prompts.			

REFERENCES

DCA Vantage Analyzer Operator’s Guide REF 06489264 Rev. B,2008-06.

Medical Approval: Dr Benjamin Jung Version: 1.2 Folder Name: CW\Point of Care\Hemoglobin A1C This is a controlled document for CW use only. Any printed copies are uncontrolled unless specified. Please refer to Lab QMS	Medical Approval Date: 28 Dec 2016 Implementation Date: 7/1/2019 11:17:25 PM
--	---

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	31 Dec 2013	Elvira Kozak	Dr. Cathy Halstead
1.1	Minor	Document title and number change. Upload to QMS document control	28 Dec 2016		Dr. Benjamin Jung
1.2	Minor	Reformatted	July 1, 2019	Calvin Lee	

Attention: This document is published on the ePOPS website

Revisions made to this document require an update to the corresponding document published on ePOPS website