

HemoCue Hb 201+: Appendix G - Training Guide and Competency Checklist

Name (Please Print):	Operator ID:	Department:	Trainer/s:	Date:	
Operator demonstrates knowledge &/or competency performance				YES	NO
Ongoing Competency – operator performs at a minimum annually:					
<ul style="list-style-type: none"> • Two [2] sets of Quality Control Solution • Two [2] Patient tests. 					
1. Reviews procedure manual, as per PHSA – C&W.					
2. Battery Change. Battery supply or AC Adaptor is available for use.					
3. External Cleaning. 2% Chlorhexidine pads, Caviwipes.					
4. Microcuvette Holder Cleaning. 2% Chlorhexidine pads, Caviwipes.					
5. Hemocue Hb 201+ Results Record - Form A: Information required.					
6. Hemocue Hb 201+ microcuvettes – Storage, expiry.					
7. Quality Control (QC) Solutions – Levels 1, 2 & 3. – Storage & open Vial Expiry					
8. Microcuvette & QC Stock Supply – Laboratory Rm. 2J8.					
9. Daily Quality Control Documents on HemoCue Hb 201+: Form A - Result Record Form					
10. Review of QC results. Discussion of corrective action when QC is out of range. Documents any action taken on HemoCue Hb 201+: Form A - Result Record Form.					
11. Proficiency Quality Assurance testing. HemoCue Hb 201+: Form A - Result Record Form. Monthly QA test – performed by Operators. Quarterly QA test - performed by Point of Care Technologist.					
12. Patient identification and confirmation HemoCue Hb 201+: Form A - Result Record Form Utilizes Institution's Clinic Patient Identifier Label when possible. (minimum of 2 patient identifiers: Name (first, last); Date of Birth; MRN or PHN.)					
13. Refers to Institutional guidelines for sample collection.					
14. Performs a simulated Hemoglobin patient test – applies whole blood: Uses the plastic tube (55uL) & applies whole blood sample to parafilm square. Bubbles in sampling will lead to a false results.					
15. To test operator reproducibility in sampling technique - simulated Hemoglobin patient test is performed in duplicate. Patient test results are within - +/- 2g/L. Document on HemoCue Hb 201+: Form A - Result Record Form					
16. Review Result Reporting HemoCue Hb 201+: Form A - Result Record Form: Hemoglobin (g/L), time, Point of Care Test Operator ID					
17. Hemoglobin Result interpretation – Appendix F - Reference Intervals.					
18. Standard precautions when performing the test. Appropriate waste disposal.					
19. Transport of documentation to POCT Laboratory – Rm. 2J8. Record Form with QC and Patient Results.					
20. Troubleshooting Guide - contact Charge RN, Educator or POCT Technologist.					

Medical Approval: De Benjamin Jung

Version: 1.1

Folder Name: CW\Point of Care\Hemoglobin

This is a controlled document for CW use only. Any printed copies are uncontrolled unless specified. Please refer to Lab QMS

Medical Approval Date: 28 Dec 2016

Implementation Date: 1/3/2017 11:36:38 AM

REFERENCES

HemoCue Hb 201+ Operating Manual. HemoCue AB, Box 1204 SE-262 23 Angelholm, Sweden.
info@hemocue.se www.hemocue.com

HemoCue Hb 201 Microcuvettes, Package Insert. HemoCue AB, Box 1204 SE-262 23 Angelholm, Sweden.
info@hemocue.se www.hemocue.com

Eurotrol HemoTrol, Assayed Controls for In Vitro Diagnostic Use, Package Insert. Eurotrol B.V. Postbus 722 6710 BS Ede, The Netherlands office@eurotrol.com www.eurotrol.com

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	Jun 2012	Elvira Kozak	Dr. Cathy Halstead
1.1	Minor	Document title and number change. Upload to QMS document control	28 Dec 2016		Dr. Benjamin Jung

Attention: This document is published on the BCCW SharePoint website

Revisions made to this document require an update to the corresponding document published on BCCW SharePoint website