
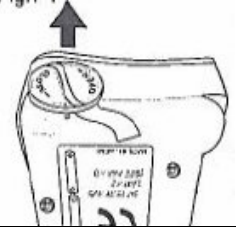

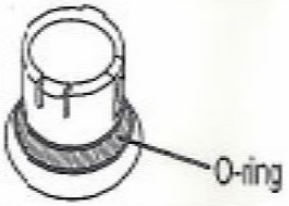
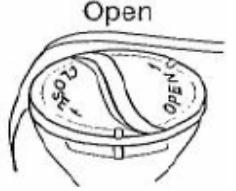
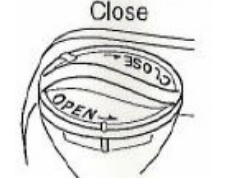


Urine Specific Gravity: Appendix E - Battery Replacement
ATAGO Pocket PAL-10S ATAGO

<ul style="list-style-type: none"> • Battery Change is indicated with the displayed error message. • 2 x 1.5 V AAA Alkaline Batteries are required. • Batteries are available on the nursing ward/units. 	
<p>1. Remove the battery case cover located on the bottom of the PAL-10S digital USG refractometer.</p>	
<p>2. Insert the two new 1.5 V AAA Alkaline batteries. Refer to the diagram on the back of the refractometer.</p>	
<p>3. Check that the O-ring located on the battery case cover is clean and free of deformities before replacing the battery case cover.</p> <p><i>Note: To prevent liquid entering the unit and causing electrical irregularities during measurement. After opening and closing the cover several times, oil (Vaseline) can be applied onto the O-ring with a cotton swab. (LAB FUNCTION ONLY).</i></p>	
<p>4. Place the battery case cover in place, press downwards, turning the cover until it stops completely.</p>	
<p>5. Confirm that the battery case cover is tightly closed in order to prevent liquids entering the unit.</p>	
<p>6. Perform a Zero Set and Urine Quality Control Check after replacing the batteries.</p>	

REFERENCES

ATAGO Instructions of Urine Specific Gravity Refractometer URIVON-Ne
Textbook of Clinical Chemistry. Norbert W.Tietz 1986 WB Saunders.

Medical Approval: Dr Benjamin Jung

Version: 1.1

Folder Name: CW\Point of Care\Urine Specific Gravity

This is a controlled document for CW use only. Any printed copies are uncontrolled unless specified. Please refer to Lab QMS

Medical Approval Date: 28 Dec 2016

Implementation Date: 1/3/2017 3:39:24 PM

REVISION & APPROVAL LOG

Version	Revision Type	Description of Change	Revision Date	Technical Approval	Medical Approval
1.0		New document	Nov 2013	Elvira Kozak	Dr. Cathy Halstead
1.1	Minor	Document title and number change. Upload to QMS document control	28 Dec 2016		Dr. Benjamin Jung

Attention: This document is published on the BCCW SharePoint website

Revisions made to this document require an update to the corresponding document published on BCCW SharePoint website